

# ichroma™ MAU Control

## [INTENDED USE]

ichroma™ MAU Control is intended for use in the clinical laboratory or doctor's office as a test sample of known concentration. Assay values are provided for the ichroma™ system or the AFIAS system in the section of "CONTROL VALUE". The user can compare test results with expected ranges as a means of assuring consistent performance of the reagent and the ichroma™ instrument or the AFIAS instrument.

- FOR IN VITRO DIAGNOSTIC USE.

## [PRODUCT DESCRIPTION]

ichroma™ MAU Control is a stable liquid control material prepared from human blood. Components of the controls, which are derived from human source materials, have been tested and no-reaction to Hepatitis B Surface antigen (HBsAg), Hepatitis C (HCV), HIV-1 and HIV-2 has been observed. However, no test method can offer complete assurance that products derived from human source are free of infectious agents.

## [INSTRUCTIONS FOR USE]

ichroma™ MAU Control is supplied in the liquid form.

1. Thoroughly mix the contents of the vial before each use by gently inverting 5~10 times.
2. Follow the procedure according to the instruction for use provided with the kit.

Dispose of any discarded materials in accordance with the requirements of your local waste management authorities. In the event of damage to the package, contact the **Boditech Med Inc.'s Technical Services**.

## [KIT COMPOSITION]

**REF** CFPO-4

Reagents (Liquid)

ichroma™ MAU Control (Vial) 1 x 0.5mL

## [STORAGE AND HANDLING]

Control should be stored refrigerated (2-8 °C). Return all reagents to 2-8 °C promptly after use. Unopened vial of control is stable until the expiration date on the label. Opened vial of control can be used for 1 month if stored at 2-8 °C.

For Technical Assistance call

**Boditech Med Inc.'s Technical Services at**

Tel: +82 (33) 243-1400

E-mail: [sales@boditech.co.kr](mailto:sales@boditech.co.kr)



**Boditech Med Incorporated**

43, Geodudanji 1-gil, Dongnae-myeon,  
Chuncheon-si, Gang-won-do. 24398

Republic of Korea

Tel: + (82) -33-243-1400

Fax: + (82) -33-243-9373

[www.boditech.co.kr](http://www.boditech.co.kr)



**Obelis s.a**

Bd. Général Wahis 53,

1030 Brussels, BELGIUM

Tel: + (32) -2-732-59-54

Fax: + (32) -2-732-60-03

E-Mail: [mail@obelis.net](mailto:mail@obelis.net)

