Document No. : INS-CH-EN (Rev. 21) Revision date : November 13, 2018





INTENDED USE

ichroma™ hsCRP is a fluorescence Immunoassay (FIA) for the quantitative determination of CRP in human whole blood/serum/ plasma. It is useful as an aid in management and monitoring of risk of cardiovascular diseases.

For in vitro diagnostic use only.

INTRODUCTION

The C-Reactive Protein (CRP) is synthesized by the liver in response to interleukin-6 and well known as one of the classical acute-phase reactants and as a marker of inflammation. It has recently been suggested that a marker of inflammation, along with serum cholesterol, may be critical component in the development and progression of atherosclerosis^{1,2}. A growing body of evidence has supported the idea that cardiovascular diseases including coronary heart disease, ischemic stroke, and acute myocardial infraction, develop, at least in part, because of a chronic low-level CRP of the vascular endothelium^{3,4}. Apparently high-sensitivity CRP (hsCRP) is emerging as the strongest and most independent predictive risk factor for atherosclerosis and CVD^{5,6}. American Heart Association (AHA) and the Centers for Disease Control and Prevention (CDC) issued a statement regarding use of C-reactive protein to assess risk of cardiovascular diseases.

PRINCIPLE

The test uses a sandwich immunodetection method; the detector antibody in buffer binds to antigen in sample, forming antigen-antibody complexes, and migrates onto nitrocellulose matrix to be captured by the other immobilized-antibody on test strip.

The more antigen in sample forms the more antigen-antibody complex and leads to stronger intensity of fluorescence signal on detector antibody, which is processed by instrument for ichroma™ tests to show CRP concentration in sample

COMPONENTS

ichroma™ hsCRP consists of 'Cartridges', 'Detection Buffer Tubes', 'Sample Collectors' and an 'ID chip'.

- The cartridge contains a test strip, the membrane which has mouse monoclonal anti-human CRP at the test line, while rabbit IgG at the control line.
- Each cartridge is individually sealed in an aluminum foil pouch containing a desiccant. 25 sealed cartridges are packed in cartridge box which also contains an ID chip.
- The detection buffer contains mouse monoclonal anti-human CRP fluorescence conjugate, goat anti-rabbit IgG fluorescence conjugate, bovine serum albumin (BSA) as a stabilizer and sodium azide in phosphate buffered saline (PBS) as a preservative.
- The detection buffer is pre-dispensed in a tube. 25 detection buffer tubes are packaged in detection buffer box and further packed in a Styrofoam box with ice-pack for the shipment.

WARNINGS AND PRECAUTIONS

- For in vitro diagnostic use only.
- Follow instructions and procedures described in this 'Instruction for use'
- Use only fresh samples and avoid direct sunlight.
- Lot numbers of all the test components (cartridge, ID chip and detection buffer) must match each other.
- Do not interchange the test components between different lots or

- pditech
- use the test components after the expiration date, either of which might yield incorrect test result(s).
- Do not reuse cartridges or detection buffer tubes. A detection buffer tube should be used for processing of one sample only. A cartridge should be used for testing one sample only.
- The cartridge should remain sealed in its original pouch until just before use. Do not use the cartridge, if pouch is damaged or has already been opened.
- Frozen sample should be thawed only once. For shipping, samples must be packed in accordance with local regulations. Sample with severe hemolysis and/or hyperlipidemia must not be used.
- Allow cartridge, detection buffer and sample to be at room temperature for approximately 30 minutes before use.
- The instrument for ichroma™ tests may generate slight vibration during use.
- Used detection buffer tubes, pipette tips and cartridges should be handled carefully and discarded by an appropriate method in accordance with relevant local regulations.
- An exposure to larger quantities of sodium azide may cause certain health issues like convulsions, low blood pressure and heart rate, loss of consciousness, lung injury and respiratory failure.
- ichroma™ hsCRP will provide accurate and reliable results subject to the below conditions.
 - Use ichroma™ hsCRP should be used only in conjunction with instrument for ichroma™ tests.
 - Have to use recommended anticoagulant sample.

| Sample type | Recommended anticoagulant | |
|-------------|-------------------------------|--|
| Whole blood | FDTA honorin Codium sitrata | |
| Plasma | EDTA, heparin, Sodium citrate | |
| Serum | Not applicable. | |

STORAGE AND STABILITY

- The cartridge is stable for 20 months (while sealed in an aluminum foil pouch) if stored at 4-30 °C.
- The detection buffer dispensed in a tube is stable for 20 months if stored at 2-8 °C.
- After the cartridge pouch is opened, the test should be performed immediately.

LIMITATION OF THE TEST SYSTEM

- The test may yield false positive result(s) due to the cross-reactions and/or non-specific adhesion of certain sample components to the capture/detector antibodies.
- The test may yield false negative result(s) due to the nonresponsiveness of the antigen to the antibodies which is most common if the epitope is masked by some unknown components, so therefore not being able to be detected or captured by the antibodies. The instability or degradation of the antigen with time and/or temperature may also cause false negative result as it makes antigen unrecognizable by the antibodies.
- Other factors may interfere with the test and cause erroneous results, such as technical/procedural errors, degradation of the test components/reagents or presence of interfering substances in the test samples.
- Any clinical diagnosis based on the test result must be supported by a comprehensive judgment of the concerned physician including clinical symptoms and other relevant test results.

MATERIALS SUPPLIED

REF CFPC-6

Components of ichroma™ hsCRP

- Cartridge Box:
 - Cartridges 25
 ID Chip 1
 Instruction For Use 1
 Sample collectors 25
- Sample collectors
 Box containing Detection Buffer Tubes
 - Detection Buffer Tubes 25

Document No. : INS-CH-EN (Rev. 21) Revision date : November 13, 2018



MATERIALS REQUIRED BUT SUPPLIED ON DEMAND

Following items can be purchased separately from ichroma™ hsCRP. Please contact our sales division for more information.

- Instrument for ichroma™ tests
 - ichroma™ Reader REF FR203
 - ichroma™ II REF FPRR021
- ichroma™ Printer REF FPRR007
- Boditech CRP Control REF CFPO-100

SAMPLE COLLECTION AND PROCESSING

The sample type for $ichroma^m hsCRP$ is $\underline{human\ whole\ blood/serum/plasma}$.

- It is recommended to test the sample within 24 hours after collection.
- The serum or plasma should be separated from the clot by centrifugation within 3 hours after the collection of whole blood. If longer storage is required, e.g. if the test could not be performed within 24 hours, serum or plasma should be immediately frozen below -20 °C. The freezing storage of sample up to 3 months does not affect the quality of results.
- In case of the whole blood sample, it should not be kept in a freezer in any case.
- Once the sample was frozen, it should be used one time only for test, because repeated freezing and thawing can result in the change test values.

TEST SETUP

- Check the contents of ichroma™ hsCRP: Sealed Cartridge, Detection Buffer Tubes, Sample Collectors and ID Chip.
- Ensure that the lot number of the cartridge matches that of the ID chip as well as the detection buffer.
- Keep the sealed cartridge (if stored in refrigerator) and the detection buffer tube at room temperature for at least 30 minutes just prior to the test. Place the cartridge on a clean, dust-free and flat surface
- Turn on the instrument for ichroma™ tests.
- Insert the ID Chip into the ID chip port of the instrument for ichroma™ tests.
- Press the 'Select' button on the instrument for ichroma™ tests.
 (Please refer to the 'Instrument for ichroma™ tests Operation Manual' for complete information and operating instructions.)

TEST PROCEDURE

< Multi mode >

- Make a puncture on the top of the detection buffer tube by inserting an empty sample collector.
- Draw 10 μL (<u>Human whole blood / serum / plasma / control</u>) of sample with a sample collector.
- 3) Assemble the sample collector and the tube into one.
- 4) Shake 10 times or more until the sample out of the sample collector by inversion. The mixture of buffer and the sample has to be used within 30 seconds.
- Remove the cap off the top of assembled tube. Discard two drops of reagent onto the paper towel before applying to the cartridge
- Load only two drops of the mixture onto the sample well of the cartridge.
- Leave the cartridge at room temperature for 3 min before inserting the device into the holder.
- holder of the instrument for ichroma™ tests. Ensure proper orientation of the cartridge before pushing it all the way inside the cartridge holder. An arrow has been marked on the cartridge especially for this purpose.

- Press 'Select' button on the instrument for ichroma™ tests to start the scanning process.
- Instrument for ichroma™ tests will start scanning the sampleloaded cartridge immediately.
- Read the test result on the display screen of the instrument for ichroma™ tests.

< Single Mode >

- Make a puncture on the top of the detection buffer tube by inserting an empty sample collector.
- Draw 10 µL (<u>Human whole blood/serum/plasma/control</u>) of sample with a sample collector.
-) Assemble the sample collector and the tube into one.
- 4) Shake 10 times or more until the sample out of the sample collector by inversion. The mixture of buffer and the sample has to be used within 30 seconds.
- Remove the cap off the top of assembled tube. Discard two drops of reagent onto the paper towel before applying to the cartridge
- Load only two drops of the mixture onto the sample well of the cartridge.
- 7) Inserting the device into the holder of the instrument for ichroma™ tests. Ensure proper orientation of the cartridge before pushing it all the way inside the cartridge holder. An arrow has been marked on the cartridge especially for this purpose.
- Press 'Select' or 'START' button on the instrument for ichroma™ tests.
- Cartridge goes inside the Instrument for ichroma™ tests and will automatically start scanning the sample-loaded cartridge after 3 min.
- Read the test result on the display screen of the instrument for ichroma™ tests.

INTERPRETATION OF TEST RESULT

- Instrument for ichroma™ tests calculates the test result automatically and displays CRP concentration of the test sample in terms of mg/L.
- The cut-off (reference value).
 - Low risk: <1.0 mg/L
 - Average risk: 1.0-3.0 mg/L
 - High risk: >3.0 mg/L
- Working range: 0.1-10 mg/L.

QUALITY CONTROL

- Quality control tests are a part of the good testing practice to confirm the expected results and validity of the assay and should be performed at regular intervals.
- The control tests should be performed immediately after opening a new test lot to ensure the test performance is not altered.
- Quality control tests should also be performed whenever there is any question concerning the validity of the test results.
- Control materials are not provided with ichroma™ hsCRP. For more information regarding obtaining the control materials, contact Boditech Med Inc.'s Sales Division for assistance.
 (Please refer to the instruction for use of control material.)

PERFORMANCE CHARACTERISTICS

Analytical Sensitivity

Limit of Blank (LoB) 0.012 mg/L
Limit of Detection (LoD) 0.029 mg/L
Limit of Quantitative (LoQ) 0.10 mg/L

Analytical Specificity

- Interference

There was no significant interference from these materials with the ichroma™ hsCRP test measurements.

Document No. : INS-CH-EN (Rev. 21) Revision date : November 13, 2018



| materials | Concentration |
|----------------------|---------------|
| Ascorbic acid | 350 µmol/L |
| Bilirubin | 350 μmol/L |
| Albumin (Protein) | 60 g/L |
| Glucose | 120 mg/dL |
| Triglyceride mixture | 500 mg/dL |
| Hemoglobin | 2,000 μg/ml |

- Cross reactivity

There was no significant cross-reactivity from these materials with the ichroma™ hsCRP test measurements.

| materials | Concentration |
|-----------------|---------------|
| CEA | 500 μg/ml |
| AFP | 100 μg/ml |
| ALT | 500 μg/ml |
| Troponin I | 500 μg/ml |
| CK-MB | 500 μg/ml |
| NT-proBNP | 500 μg/ml |
| Serum amyloid P | 500 µa/ml |

Precision

- Between Lot

One person tested three different lots of **ichroma™ hsCRP**, five times at each concentration of the control standard.

- Between person

Three different persons tested ichroma™ hsCRP, five times at each concentration of the control standard.

Between day

One person tested **ichroma**TM **hsCRP** during five days, five times at each concentration of the control standard.

- Between site

Three persons tested **ichroma™ hsCRP** at three different sites, five

| | CIIII | es at ea | cii conc | ciitiatio | iii oi tiie | CONTRO | Stanuai | u. | |
|---|--------|-------------|----------|----------------|-------------|-------------|---------|--------------|-------|
| Ī | Conc. | Between-lot | | Between-person | | Between-day | | Between-site | |
| ı | [mg/L] | AVG | CV(%) | AVG | CV(%) | AVG | CV(%) | AVG | CV(%) |
| | 0.5 | 0.48 | 3.53 | 0.49 | 4.71 | 0.48 | 3.84 | 0.49 | 3.80 |
| | 1.5 | 1.48 | 3.53 | 1.47 | 4.26 | 1.49 | 3.42 | 1.49 | 3.78 |
| • | 5 | 5.01 | 5.56 | 4.90 | 8.50 | 5.03 | 4.79 | 5.03 | 3.03 |


Accuracy

The accuracy was confirmed by 3 different lots testing ten times each different concentrations.

| CRP [mg/L] | Lot 1 | Lot 2 | Lot 3 | AVG | Recovery (%) |
|------------|-------|-------|-------|------|--------------|
| 0.5 | 0.47 | 0.49 | 0.49 | 0.48 | 97% |
| 1.5 | 1.48 | 1.47 | 1.50 | 1.48 | 99% |
| 5 | 5.12 | 5.04 | 5.12 | 5.09 | 102% |

Comparability

CRP serum samples were quantified independently with ichroma™ hsCRP and Hitachi 7020 (Hitachi Inc. Japan) as per prescribed test procedures. Test results were compared and their comparability was investigated with linear regression and coefficient of correlation (R). Linear regression and coefficient of correlation between the two tests are as follows respectively.

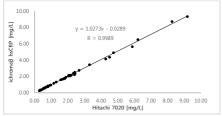

REFERENCES

- Koenig W, Sund M, Frohlich M, et al. C-reactive protein, a sensitive marker of inflammation, predicts future risk of coronary heart disease in initially healthy middle-aged men. Circulation 1999; 99:237-242.
- Rifai N, Ridker PM. Proposed Cardilvascular Risk Assessment Algorithm Using High-Sensitivity C-Reactive Protein and Lipid Screening. Clin Chem 2001; 47:28-30.
- Rifai N and Ridker PM. High-Sensitivity C-Reactive Protein: A novel and Promising Marker of Coronary Heart Disease. Clin Chem 2001; 47(3):403-411.
- Biasucci LM, Liuzzo G, Grillo RL, et al. Elevated levels of C-reactive protein at discharge in patients with unstable angina predict recurrent instability. Circulation 1999;99:855-860.
- Taubes G. Does inflammation cut to the heart of the matter? Science 2002; 296:242-245.
- Ridker PM, Hennekens CH, Buring JE, and Rifai N. C-reacitve protein and other markers of inflammation in the prediction of cardiovasculare disease in women. N Engl J Med 2000:342(12): 836-843
- Brooks DE, Devine DV, Harris PC, et al. RAMP(TM): A rapid, quantitative whole blood immunochromatographic platform for point-of-care testing. Clin Chem 1999; 45:1676-1678.
- Oh SW, Moon JD, Park SY, et al. Evaluation of fluorescence hs-CRP immunoassay for point-of –care testing. Clin Chim Acta 2005; 356:172-177.

Note: Please refer to the table below to identify various symbols

| Σ | Sufficient for <n> tests</n> |
|-------------|---|
| (]i | Read instruction for use |
| \square | Use by Date |
| LOT | Batch code |
| REF | Catalog number |
| \triangle | Caution |
| 444 | Manufacturer |
| EC REP | Authorized representative of the European Community |
| IVD | In vitro diagnostic medical device |
| X | Temperature limit |
| (2) | Do not reuse |
| CE | This product fulfills the requirements of the Directive 98/79/EC on in vitro diagnostic medical devices |

For technical assistance; please contact:

Boditech Med Inc.'s Technical Services

Tel: +82 33 243-1400 E-mail: sales@boditech.co.kr

Boditech Med Incorporated

43, Geodudanji 1-gil, Dongnae-myeon, Chuncheon-si, Gang-won-do, 24398 Republic of Korea

Tel: +(82) -33-243-1400 Fax: +(82) -33-243-9373

Fax: +(82) -33-24

EC REP Obelis s.a

Bd. Général Wahis 53, 1030 Brussels, BELGIUM

Tel: +(32) -2-732-59-54 Fax: +(32) -2-732-60-03 E-Mail: mail@obelis.net

